

# Accusol

## Dialysis solution for CRRT



### Clear and simple.

In CRRT, it is essential for patients to be cared for quickly and efficiently. ACCUSOL dialysis solution for CRRT offers health care providers a convenient and easy-to-use alternative that provides confidence and flexibility. ACCUSOL dialysis solution contains 100% bicarbonate as a buffer and is completely lactate-free.

It is an innovative, simplified, and safe means of providing quick and efficient care for CRRT patients.

### Ease of use.

- Simple, two-step mixing process – takes only seconds of hands-on time.
- Easy-to-handle 2.5 L bags
- Ready to connect and use quickly

100% bicarbonate buffer, **lactate free**


**Five** formulations

**Quick** two-step mixing

Individualized Prescribing.

# Accusol

## Dialysis solution for CRRT

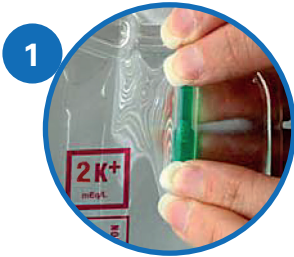

**Step 1:** Hang bag.  
Snap green frangible seal.



**Step 2:** Wait for the upper  
chamber to completely drain.  
Agitate gently. Solution is ready.

### ACCUSOL dialysis solution.

100% bicarbonate buffer, lactate-free.

Product name		ACCUSOL 35 K4/Ca 3.5	ACCUSOL 35 K2/Ca 3.5	ACCUSOL 35 K0/Ca 3.5	ACCUSOL 30 K2/Ca 2.8	ACCUSOL 30 K2/Ca 0
Product code		5B9248	5B9249	5B9250	5B9251	5B9252
Sodium	mEq/L	140	140	140	140	140
Potassium	mEq/L	4	2	0	2	2
Bicarbonate	mEq/L	35	35	35	30	30
Chloride	mEq/L	113.5	111.5	109.5	116.3	113.5
Calcium	mEq/L	3.5	3.5	3.5	2.8	0
Magnesium	mEq/L	1.0	1.0	1.0	1.5	1.5
Dextrose anhydrous	mg/dL	100	100	0	100	100
pH		7.4	7.4	7.4	7.4	7.4

Volume 2500 mL

### ACCUSOL dialysis solution's features and unique compositions assure safety, efficiency, and flexibility.

- **Individualized prescribing.** Five ready-to-mix formulations offering variable bicarbonate and multiple potassium concentrations, better matching your patient prescription.
- **100% bicarbonate buffer.** ACCUSOL dialysis solution contains all of the necessary electrolytes in a wide range of formulations.
- **Lactate-free.** Eliminating the need to use lactate as a stabilizing agent, ACCUSOL dialysis solution is prime for treating patients when the use of lactate should be avoided.
- **Clearly labeled.** Potassium concentrations are clearly placed in three locations on the bag, one of them in brilliant red.
- **Confidence in solution flow.** Green frangible seal remains stationary and is designed to not detach during use.

To learn more, call us at (858) 222-6300 or email us at [info@nikkisoamerica.com](mailto:info@nikkisoamerica.com)

Distributed by:  
Nikkiso America, Inc.  
5910 Pacific Center  
Boulevard # 110  
San Diego, CA 92121  
U.S.A.

Manufactured for:  
Nikkiso Belgium bvba  
B-3300 Tienen  
Belgium

Made in Canada

Caution: Federal (US) law restricts these devices to sale by or on the order of a physician or other licensed practitioner.

For safe and proper use of these devices refer to the device instructions.

Warning: For use as a dialysis solution only. Not for direct intravenous infusion.

ACCUSOL is a trademark of Nikkiso Co., Ltd.

PM-0105-06/2016-1 Copyright 2017. All rights reserved.

[www.nikkisoamerica.com](http://www.nikkisoamerica.com)